# ClearNote

User manual  
Gebruikershandleiding  
Bedienungsanleitung  
Manual d'emploi  
Manuale utente

**User manual for ClearNote**  
**Gebbruikershandleiding voor ClearNote**  
**Bedienungsanleitung für ClearNote**  
**Manuel d'emploi pour ClearNote**  
**Manuale utente per ClearNote**

**User manual version 1.2**  
**Copyright 2008 Optelec, the Netherlands.**

**All rights reserved**

Optelec  
P.O. Box 399  
2990 AJ Barendrecht  
The Netherlands  
Telephone: +31 (0)88 6783 444. Fax +31 (0)88 6783 400  
E-mail: [info@optelec.nl](mailto:info@optelec.nl)  
Internet: [www.optelec.com](http://www.optelec.com)

# **ClearNote User manual**

## **For ClearNote with software version 7**

October 2008

Optelec, part of Tieman Group, was founded in 1975 in Rockanje, the Netherlands. Since then, Optelec has developed and produced innovative products to assist blind and visually impaired people in their daily activities. Visit the Optelec website at [www.optelec.com](http://www.optelec.com) for extensive information about Optelec products and services.

All rights, in particular the rights of reproduction as well as translation, are reserved. No parts of this manual may be copied, in any form whatsoever, without written approval by Optelec.

Every effort has been made to ensure that the information contained in this user manual is correct at the time of issue. Neither Optelec, nor its agents, assume any responsibility for errors or damages resulting from use other than described in this manual.

# Index

1.	Introduction.....	3
	About this manual .....	3
2.	Important safety information .....	4
3.	What's in the box? .....	5
4.	Installing the ClearNote software.....	6
5.	Getting to know your ClearNote .....	7
	Assembling the ClearNote.....	7
	Using ClearNote with a laptop .....	7
	Positioning the camera head.....	7
	Camera positions .....	7
	Close-up and distance viewing.....	7
6.	Operation with the remote control .....	8
	Basic functions .....	8
	Setting the magnification .....	8
	Viewing modes .....	8
	Semi-colours .....	9
	Autofocus .....	9
	Shift .....	9
	Shift functions .....	9
	Manual focus .....	10
	Positioning the reading line or the split-screen.....	10
	Switching between open programs .....	10
7.	Operation ClearNote with the computer keyboard .....	11
	Starting the software .....	11
	List of commands .....	11
	Keyboard functions.....	11
	Setting the magnification .....	11
	Viewing modes .....	11
	Semi-colours .....	11
	Resizing and moving the ClearNote window (Picture in Picture) .....	11
	Activating and sizing line markers .....	12
	Save and open images.....	12
	Mirror/flip the image.....	12
	Brightness, contrast and colour saturation of the image .....	12
8.	ClearNote and magnification software.....	13
9.	ClearNote specifications.....	14
10.	Compliances.....	15

# 1. Introduction

Congratulations on choosing the ClearNote, a portable and foldable video magnifier designed by Optelec. ClearNote is an auto-focus video magnifier designed to assist people with low vision on different locations. ClearNote together with a computer enables reading, writing, viewing photos, craftwork and lots more. The ClearNote magnifier is foldable which means that it can be folded into a compact size to take it with you comfortably.

The ClearNote offers an enhanced ergonomic design that allows simple operation. The ergonomics are designed for optimal efficiency when working with a computer. The ClearNote can be operated directly from the keyboard of your desktop or laptop computer or via the remote control.

ClearNote software is compatible with the Windows XP Service Pack 2 or Windows Vista operating systems. The unit should be connected via an USB 2.0 port.

If you have any questions or suggestions concerning the use of this product, please contact your distributor or the Optelec headquarters using the contact information at the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your ClearNote.

## About this manual

This manual will familiarize you with the features and operation of the ClearNote and ClearNote software with a computer. Please read this manual carefully before using your device. Keep this manual in a safe place for future reference.

## 2. Important safety information

- Only use the ClearNote indoors.
- Do not expose the ClearNote to extreme or rapid changes in temperature or humidity. These conditions may damage the unit's electronics or optics.
- Do not place the unit on an unstable surface. It may fall, causing serious damage to the unit or injuries to the user.
- To avoid injuries always assemble, disassemble and handle the unit with good care as the unit contains movable and rotatable parts.
- Do not expose the ClearNote to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the ClearNote. Contact your Optelec distributor to service the ClearNote if necessary. Do not open your unit as this will void the warranty. The unit contains no user-serviceable parts.
- To avoid risk of electrical damage, keep your ClearNote away from fluids and chemicals
- Handle the ClearNote with care. Rough handling will damage the internal components
- Do not use the ClearNote near inadequately shielded medical devices.
- Always unplug the unit before cleaning. Use a soft damp cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage the unit.
- Do not use the unit under +0° or above +40° Celsius.

Use of the ClearNote other than described in this manual will exclude it from warranty conditions.

### 3. What's in the box?

The ClearNote packaging contains the following:

- A carrying bag
- The ClearNote video magnifier
- Base plate
- Remote control
- The ClearNote software and documentation on CD-ROM
- User manual
- Hotkey lists

If any of these items are missing from your package, please contact your local distributor.

## 4. Installing the ClearNote software

**Important:** Install the ClearNote software before connecting your unit to a computer.

The ClearNote software is necessary to use the ClearNote with your computer. Therefore the ClearNote software and image reproduction program must be installed on your laptop or desktop computer.

The installation can be completed in the following steps:

1. Make sure that the ClearNote USB cable is not connected to your computer.
2. Insert the ClearNote software CD-ROM that came with the package in your computer's disk drive.
3. The program will automatically run if the autostart feature of your CD-ROM drive is enabled.
  - a. If the autostart feature has not been enabled, open your Windows Start Menu and select 'Run'.
  - b. Alternatively you can double click the CD drive icon in 'My Computer'.
  - c. Select your preferred language and click "Next".
  - d. Click "Next" in the 'Welcome to Setup Wizard'.
  - e. Choose the destination folder and click "Next".
  - f. Select the software magnification program that runs on your computer. If you have not installed magnification software, please select "default" installation.
  - g. Click "Next" in the 'Select Start Menu folder'.
  - h. Select "Install the camera driver" and choose your preferred desktop icons;
  - i. Click on "Install";
  - j. Click on "Finish" to complete the setup.
4. Now connect the ClearNote USB cable to a USB ports on your computer;
5. The screen will say "new hardware found". Install the USB driver by selecting "do not look for updates" when asked on the screen. In the next screen choose for "automatic installation of the USB driver".
6. If the "files needed" screen appears looking for a driver, it is in: C:\Program files\ClearNote 7\driver, unless you installed it in another subfolder.
7. Repeat the actions from step 6 till you installed the ClearNote for all possible USB ports on your computer.
8. Start the Software by clicking the ClearNote 7 icon on your desktop or in the Windows Start Menu.

When carrying out the standard installation, you will find the program files under C:\Program Files\ClearNote 7

## 5. Getting to know your ClearNote

### Assembling the ClearNote

**Important:** Install the ClearNote software before you connect your unit to a computer. Refer to chapter 5 for information on how to install the ClearNote software

To assemble your ClearNote unit please follow the steps described below:

1. Place the base plate nearby your computer on a steady surface.
2. Unfold the ClearNote unit.
3. Place the ClearNote on the connection point of the base plate.
4. Connect the remote control to the camera stand.
5. If you have already installed the ClearNote software connect the ClearNote output cable (USB 2.0) to your desktop or laptop computer. To read about the installation of the ClearNote software please read the next chapter.

### Using ClearNote with a laptop

Place your laptop on the ClearNote base plate and connect the USB cable in your computer's USB 2.0 port. The ClearNote can be positioned either left or right of your laptop or desktop PC.

### Positioning the camera head

ClearNote features an ultra-flexible camera that can be rotated around a horizontal and vertical axis. Point the camera to a document for close-up viewing. Alternatively, position the camera horizontally for distance or self-viewing. The flexible camera head approaches a view of approximately 360°.

### Camera positions

The ClearNote automatically re-adjusts to its original settings when the camera is turned from near to distance viewing and back. For example when you tilt the camera quickly from a whiteboard to your desktop, your preferred viewing mode and magnification settings will automatically be applied. Three sets of settings can be stored: document viewing, distance viewing and looking behind you.

### Close-up and distance viewing


The position of the camera unit can be adjusted for desktop or distance viewing. To view objects within 25 cm, position the close-up lens in front of the camera unit. To view objects at a larger distance, position the close-up lens away from the camera.

## 6. Operation with the remote control

To use the ClearNote application make sure to connect the ClearNote to the computer via the USB cable and start the ClearNote software.

Click the close (x) button in the upper right corner of the ClearNote window to close the ClearNote application.

### Basic functions

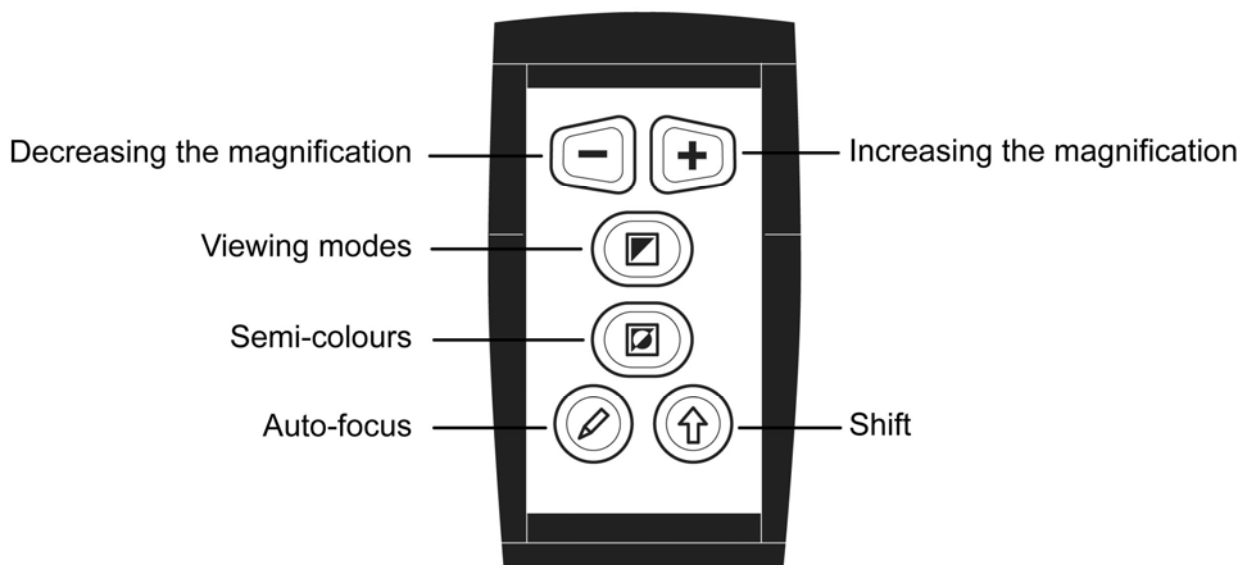### Setting the magnification

Pressing the two trapezium shaped buttons allows you to select the magnification levels. ClearNote magnifies text and images from 4 to 40 times the original size. Pressing the “-” button will decrease the magnification. Pressing the “+” button will increase the magnification.

### Viewing modes

Pressing this button will cycle you through 3 viewing modes:

#### 1- Photo mode Full colour

This mode will display texts, pictures and objects in full colour.

#### 2- Reading mode

This mode enhances the foreground / background contrast. Pictures and text will be displayed in black text and white background.

#### 3- Inverse Reading mode

This mode inverses the chosen background and foreground colour combinations. Pictures and text will be displayed in white text on a black background.

## Semi-colours

You can activate this function in the Reading modes only. Pressing this button cycles you through 10 different foreground and background colour combinations. You may choose to display text in a pair of selectable colours when more contrast is needed or when alternate colour combinations provide better visibility.

## Autofocus

The ClearNote features automatic focusing producing a clear image at any time. Pressing the Autofocus button stops the ClearNote from continuously focusing. This is useful for tasks such as writing, painting or enjoying craftwork. Pressing the auto-focus button again will re-activate autofocus.


## Shift

Pressing the Shift button will activate the ClearNote window. Using this button in combination with other remote control buttons will result in extra operating functionality. Read the following page about the Shift functions.

## Shift functions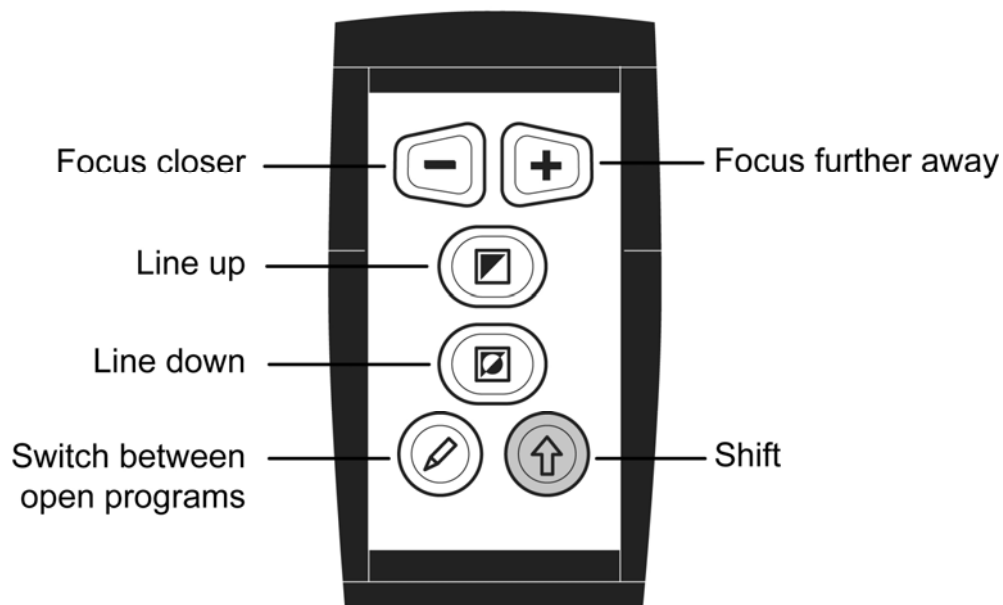

## Manual focus

To adjust the camera focus manually, switch the Autofocus off and press the “Shift” and the “Magnification buttons” simultaneously. Pressing “Shift” in combination with the “-” button allow a closer focus, whereas pressing the “Shift” button in combination with the “+” button, will make the ClearNote focus further away.

## Positioning the reading line or the split-screen

Press the “L” key on the computer keyboard to add a reading line to the ClearNote image. Pressing the “Shift” and “Viewing mode” button simultaneously enables moving the line up. With pressing the “Shift” and “Semi-colours” button you move the line down.

If the ClearNote is used simultaneously with a magnification software (chapter 8) the ClearNote appears in a split-screen format. In this case pressing the “Shift” and “Viewing mode” button simultaneously enables moving the split up. With pressing the “Shift” and “Semi-colours” button you move the split down.

## Switching between open programs

Pressing the “Shift” and the “Autofocus” button simultaneously enables you to change between the applications that are running on your computer.

## 7. Operation ClearNote with the computer keyboard

### Starting the software

Start the ClearNote application by clicking either the ClearNote icon on your computer's desktop or via the Windows Start Menu.

You can close the ClearNote application as any other Windows program using "Alt+F4" or by clicking the close (x) button in the upper corner of the ClearNote window.

### List of commands

For a description of the keyboard functions, please refer to the included hotkey list.

Whilst operating ClearNote on your PC, the hotkey list can also be displayed on the screen pressing "H". To close this screen again, press "Alt+F4".

### Keyboard functions

#### Setting the magnification

Pressing the "Arrow down" button will decrease the magnification. Pressing the "Arrow up" button will increase the magnification.

#### Viewing modes

Pressing the "M" button on the keyboard will cycle you through 3 different viewing modes: Full colour photo mode, high contrast positive and negative reading modes.

#### Semi-colours

You may choose to display text using a range of high-contrast semi-colour combinations. These can be activated by pressing "S". With "Shift+S" you can cycle through each of the 10 different foreground and background colours. The colour combinations can be inverted by pressing "M".

#### Resizing and moving the ClearNote window (Picture in Picture)

The size of the ClearNote window can be adjusted using the mouse or hotkeys. If you change the window size from full screen, you can move it to each corner of the screen pressing the "W" successively.

Press "F" to maximize the ClearNote window to full screen or to restore it down.

The size can be decreased and increased in small steps by pressing "Ctrl + one of the four arrow keys".

The ClearNote window can be moved in either direction of your screen by pressing “Shift + one of the four arrow keys.”

The ClearNote window can be placed on the foreground or background of your active programs by pressing “Alt+Tab” successively.

### **Activating and sizing line markers**

Reading lines can be used for better orientation or as a reference when using the ClearNote. Press “L” to activate this function. Use the “Page up” and “Page down” to move the line up and down.

### **Save and open images**

The ClearNote software enables image saving. Pressing the “Shift” key in combination with number keys (0-9)”, allows you to save up to 10 images in a JPG format. The images will be saved in the ClearNote folder in “My Documents”.

Images saved with “Shift + number keys” will be automatically named according to the pressed number key (e.g. pic\_1.jpg). When saving a new image the content of the existing pic\_x.jpg files will be overwritten. Press “Ctrl+S” to rename the images.

Pressing “Ctrl+O” enables opening the ClearNote folder and the saved images. Press the number keys “0-9” to open the images directly.

### **Mirror/flip the image**

Pressing “Shift+M” allows you to horizontally mirror/flip the video image. To mirror the image vertically press “Ctrl+M”. You may choose this function when tilting the camera results an image upside down or for self-viewing.

### **Brightness, contrast and colour saturation of the image**

In order to optimise the displayed image of the ClearNote, you can make the following adjustments:

- Increase brightness : “B”
- Reduce brightness : “Shift+B”
- Increase contrast : “C”
- Reduce contrast : “Shift+C”
- Increase colour saturation : “X”
- Reduce colour saturation : “Shift-X”
- Reset to factory defaults : “Ctrl+R”

## **8. ClearNote and magnification software**

ClearNote can be used simultaneously with the magnification software programs such as Lunar(Plus), SuperNova and ZoomText. The ClearNote image appears adjacent to the magnified Windows application(s), appearing split-screen format. The ClearNote image can be re-sized and positioned anywhere on your screen. Other application(s) are automatically adjusted.

To make use of this function you need a Dolphin magnification software package like Lunar(Plus) or SuperNova (version 7.01 or higher) or ZoomText (version 9.04 or higher).

Please note to start your magnification software first, followed by the ClearNote application. When closing down, make sure to close the ClearNote window first, followed by the magnification software.

## 9. ClearNote specifications

- Camera : 10x optical zoom
- Viewing modes : Full colour photo mode  
High contrast black-on-white  
High contrast white-on-black
- Semi-colours : 10 foreground / background  
colour combinations
- Magnification : 4 to 40 times on a 17" monitor
- Focus : Autofocus with focus lock and manual  
focus
- Weight : 1,1 kg / 39 ounces
- PC requirements : Windows XP SP2 or Windows Vista
- PC specifications : Intel Pentium IV 1.6Ghz or higher  
(recommended)  
32 MB video RAM minimum  
(64 recommended)
- Connections : USB 2.0

### Operating conditions

- Humidity : < 75%, no condensation
- Temperature : 0 – 40 °C

### Storage & transport conditions

- Humidity : < 75%, no condensation
- Temperature : -20 – 60 °C

**WARNING:** The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by the manufacturer of the ClearNote as replacement parts for internal components, may result in increased emissions or decreased immunity of the ClearNote.

Optelec nor its partners do not take any responsibility when using the product and its accessories other than those specified in this manual.

For additional questions on the ClearNote regarding service, installation or operation, do not hesitate to contact your local distributor or closest Optelec office. See the last pages of this manual for a complete list of Optelec addresses.

## 10. Compliances

This product complies with the safety and EMC requirements of the directives:

EN60601-1 : 2006

EN60601-1-2 : 2007

### **Optelec**

Breslau 4

2993 LT Barendrecht

The Netherlands

### **EU – Declaration of conformity**

Barendrecht, the Netherlands

October 2008

The manufacturer declares that the:

Video magnifier **ClearNote** complies to the safety guidelines and directives of the European Community.

This EU – declaration of conformity is based on tests according to the directives:

EN 60601-1:2006 and EN 60601-1-2:2007 by an independent Notified body.

**This device bears the CE approval mark**

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**Tieman GmbH**
**Optelec US Ltd**
U.S.A.