MultiView

User Manual

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MultiView

User Manual	English
Bedienungsanleitung	Deutsch
Manuel d'utilisation	Français
Gebruikershandleiding	Nederlands
Manuale utente	Italiano
Manual del usuario	Español

Version 1.4

Copyright 2009 - Optelec, the Netherlands

All rights reserved

Optelec
P.O. Box 399
2993 LT Barendrecht
The Netherlands
Telephone: +31 (0)88 6783 444. Fax +31 (0)88 6783 400
E-mail: info@optelec.nl
Web site: www.optelec.com

MultiView User Manual

Optelec, part of Tieman Group, was founded in 1975 in Rockanje, the Netherlands. Since 1975, Optelec has been developing and producing innovative products to assist blind and vision-impaired people in their daily activities.

Visit the Optelec web site at www.optelec.com for more information about Optelec's products and services.

Every effort has been made to ensure that the information contained in this user manual is correct at the time of its publication. Neither Optelec, nor any of its associates, shall not be held responsible for any errors or damages that may result from the incorrect use of the information contained in this manual.

Index

1.	Introduction	5
	About this manual	5
2.	Safety and Care	6
3.	What's in the Box?	7
4.	Getting to Know Your MultiView.....	8
	Setting up the MultiView	8
	Choosing between left- or right-hand usage.....	9
	Positioning the camera head and camera arm	9
	Camera positions.....	10
	Close-up viewing	10
	Self-viewing	11
	Distance viewing.....	11
	Using the reading table.....	11
5.	Getting Started	12
	Switching On / Off.....	12
	Setting the level of magnification	12
	Viewing modes	12
	Find function	13
	Autofocus lock	13
	Using the Snapshot button	13
	Activating line markers / window shades	13
	Positioning line markers / window shades	13
	White Balance	14
	Left / right rotation.....	14
6.	Contact Information / Product Support.....	15
7.	Warranty Conditions.....	15
8.	Compliances	16
9.	Specifications of the MultiView.....	17

1. Introduction

Congratulations on choosing the MultiView, an easily-transportable autofocus video magnification system from Optelec. Use the MultiView for reading, writing, viewing photographs, doing crafts, viewing distant objects, putting on makeup, and so much more!

The MultiView enables you to view printed text, images and objects in different colors, and to magnify them up to 79 times on a 19" wide screen display. With its ergonomic control pad, flexible camera arm and light-weight X-Y table the MultiView extends in operating and reading comfort. The MultiView can be used at home or in school and office settings.

If you have any questions or suggestions concerning this product, please contact your distributor or the Optelec headquarters by consulting the contact information page found at the end of this user manual. Your feedback is greatly appreciated. We hope you enjoy working with your MultiView.

About this manual

This user manual offers a comprehensive guide to the features and operation of the MultiView video magnifier. The illustrations in this user manual may be different from the actual situation. Optelec will not assume any responsibility for the misinterpretation of any of the content found in this user manual. Please read this user manual carefully before operating your MultiView, and keep it handy at all times for immediate reference.

2. Safety and Care

- Only use the MultiView indoors.
- Do not expose the MultiView to extreme or rapid changes in temperature or humidity. Do not place the MultiView near a radiator or heater, or where it may be exposed to direct sunlight. These conditions may damage the MultiView's electronic or optical components.
- Do not use the MultiView near other electrical appliances or inadequately-shielded medical devices as this could cause electromagnetic interferences.
- Do not place the MultiView on an unstable surface as it may result in a fall, causing serious damage to the unit or injury to the user.
- Handle the MultiView with care. Any rough handling will damage its internal components.
- When moving the MultiView to another location, disconnect the power cable along with all other cables from all external connectors. To carry the MultiView in a safe and easy manner, use the provided handle located at the back of the unit.
- The reading table and camera arm feature moving and removable mechanical parts. Please ensure your safety when removing or adjusting these mechanical parts (users are not recommended to remove these mechanical parts frequently). Injury could occur if fingers or loose clothing become caught in the operating mechanisms. If you experience any difficulties operating the unit safely, please ask for help by contacting your local Optelec distributor.
- Do not remove any parts from the MultiView. Contact your local Optelec distributor to service your MultiView, if needed. Do not open any of the components as this will void the warranty. Each component contains parts which should only be serviced by an Optelec repair specialist.
- Always turn the power off when the system is not in use.
- Always unplug the system before cleaning it. Use a soft, damp cloth to clean its external parts. Do not use cleaning agents or abrasive materials as they will damage your unit. Avoid touching the lens of the camera and keep it clean by using proper cleaning means.
- Only use the supplied power cable.
- Do not use the unit in temperatures under +5°C and above +40°C.

Any use of the MultiView other than what is described in this user manual will exclude it from warranty conditions.

3. What's in the Box?


The MultiView packaging contains the following items:

- The MultiView video magnifier
- A detachable camera unit
- A control pad
- An X-Y reading table
- A power cable
- This user manual
- A carrying case (optional)

If any items are missing from your package, please contact your local Optelec distributor.

4. Getting to Know Your MultiView

The following figure shows the MultiView's main components.

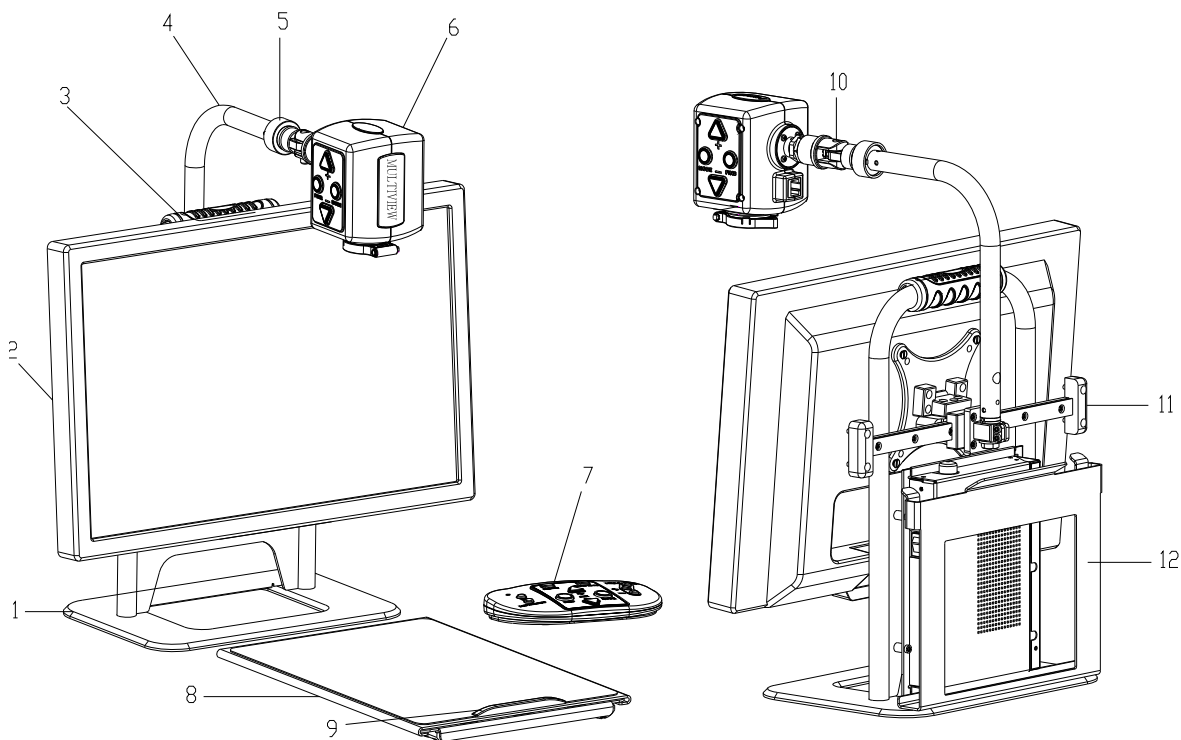
- | | |
|----------------------------|--------------------------------|
| 1. Base stand | 7. Control pad |
| 2. 19" wide screen display | 8. X-Y reading table |
| 3. Handle | 9. Handle of X-Y reading table |
| 4. Camera arm | 10. Camera rotation axis |
| 5. Camera connection point | 11. Slide track |
| 6. Camera head | 12. Reading table support |

Setting up the MultiView

To set up the MultiView, please follow the steps described below:

- Install the camera head:
 - Attach the camera head to the camera arm by fastening the screw at the camera connection point.
 - Connect the data cable to the data cable connector located in back of the camera head.
- Connect the control pad in the USB port located on the left side of the base stand.
- Connect the power cable in the power connector located on the right side of the base stand.
- Plug the other end of the power cable into an electrical power outlet.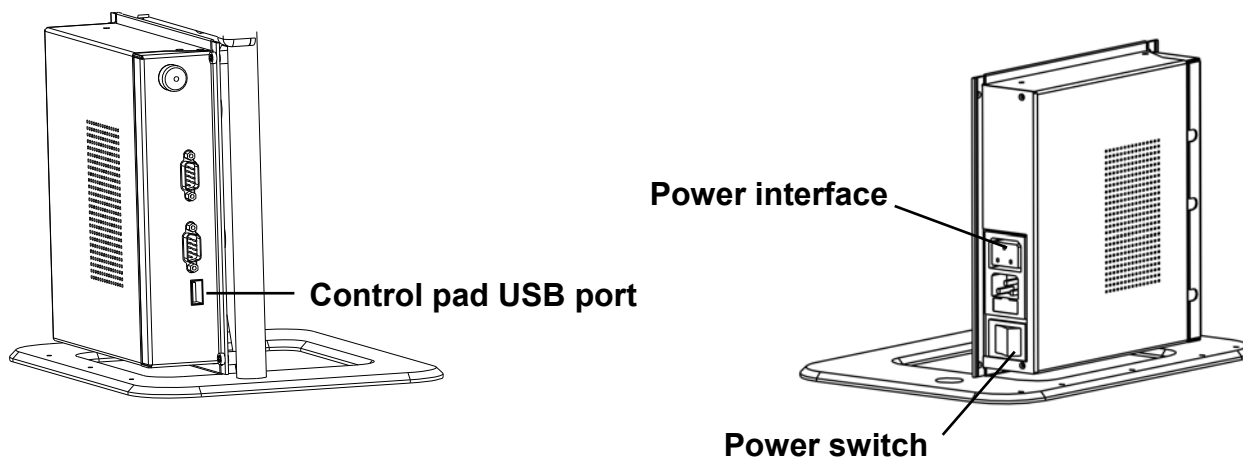


Choosing between left- or right-hand usage

The MultiView was designed for both left- and right-hand users. To set the unit according to the type of user, position the camera head as described in the next paragraph and rotate the MultiView image, if necessary, by pressing the left / right hand use button .


Positioning the camera head and camera arm

The MultiView features a flexible camera head and camera arm. The camera head can be rotated 180 degrees at a vertical axis in front of the screen to suit either the left- or right-handed user. The following figure shows the camera head in the position for right- and left-handed users.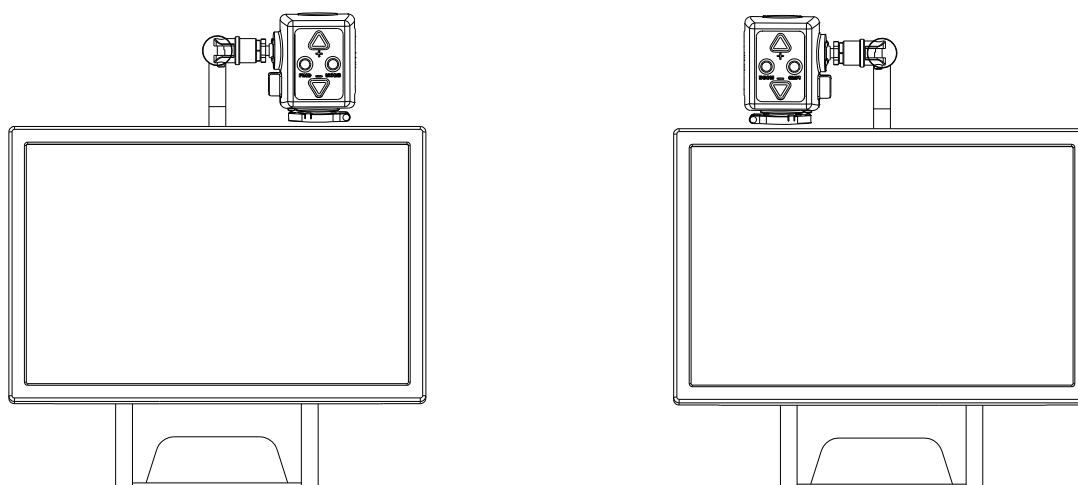


Camera position for right-handed user Camera position for left-handed user

At a horizontal axis, the camera head can be tilted 270 degrees. Tilt the camera head and rotate it to view objects further away such as a whiteboard. The following figure shows the different tilt movements.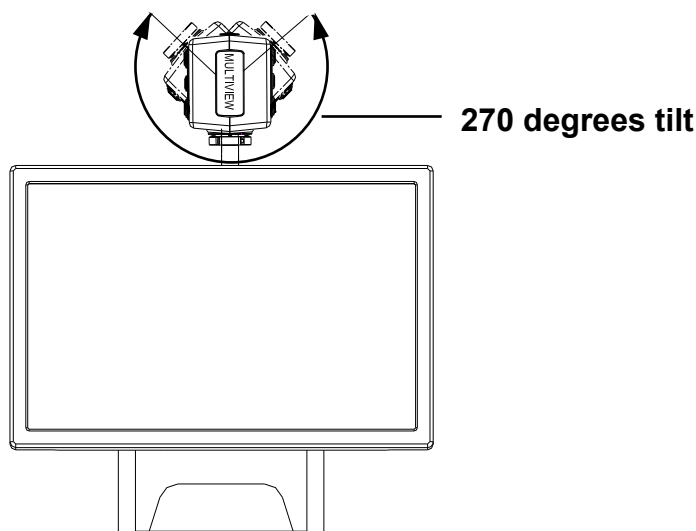

Note: Do not force the camera head beyond its limit as this could permanently damage the rotation axis feature.

The camera arm can be moved from left to right, and right to left, by using the slider located at the back of the base stand. The following figure shows the slider.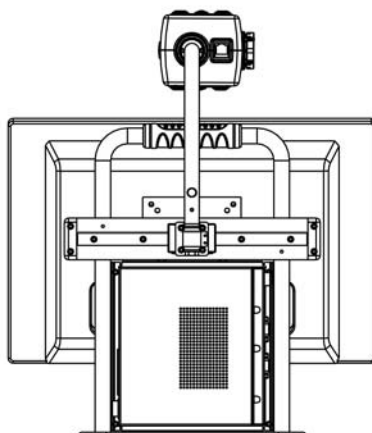


Camera positions

The MultiView automatically returns to its original settings when the camera is switched from close-up to distance viewing and vice-versa. For example, when the camera is tilted quickly from the whiteboard you are viewing to the top of your desk, your preferred viewing mode and magnification settings will automatically be applied. Three setting options can be stored: close-up viewing, self-viewing and distance viewing.

Close-up viewing

To view objects placed on a table, or to write under the camera, point the camera head toward the top of the desk and position the close-up lens in front of the camera. The recommended distance for close-up viewing is between 30 and 40 cm (12 and 16 inches).

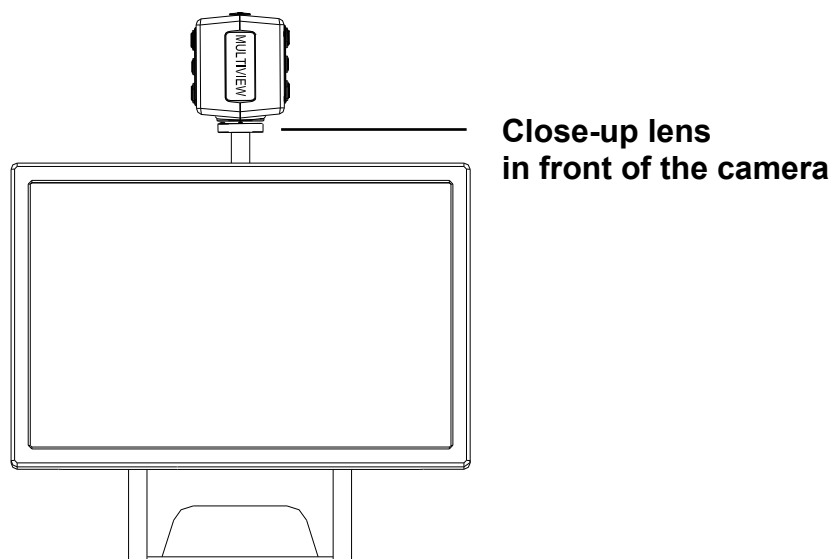Self-viewing

To view yourself and to use the MultiView as a mirror, point the camera toward you and position the close-up lens away from the camera. The recommended distance for self-viewing is 40 to 50 cm (16 to 20 inches).

Distance viewing

To view objects further away at a distance between 1 and 5 m (3 and 15 feet), point the camera head toward the object and remove the close-up lens. For instance, this viewing mode can be used to read text on a whiteboard.

Using the reading table

The X-Y reading table is used to easily move a document and read its text from left to right, and up and down in a straight line.

Place the reading table in front of you as you like it, either horizontally or vertically, and align the document you wish to read on the table. To start reading, use the table's handle to move the document as you read it. The guided table movement will prevent you from losing of the text.

After you have finished reading, the table can be easily rolled under the display in a horizontal position as to gain surface space on your desk.

5. Getting Started

Before operating the MultiView, the camera head needs to be installed.

Switching On / Off

Press the On / Off switch located on the right side of the base unit to power the MultiView.

Setting the level of magnification

Press the “+” and “-” button located at the centre of the control pad, or on the camera head to adjust the level of magnification. To increase the magnification, press the “+” button and to decrease it, press the “-” button. To adjust the level of magnification in a continuous manner, press and hold down either buttons for a few seconds. The MultiView can magnify text and objects from 3.5 to 79 times their original size (on a 19” wide screen LCD screen) and at a surface distance of 40 cm away from the camera.

Viewing modes

Pressing this button on the control pad or on the camera head will cycle you through 8 different viewing modes:

- **Full Color Photo Mode**

This mode will display text, pictures and objects in full color.

- **Gray Scale Photo Mode**

This mode will display text, pictures and object in shades of gray.

- **Positive Reading Mode (High contrast black on white)**

This mode enhances the foreground / background contrast. Pictures and text will be displayed as black text on a white background.

- **Negative Reading Mode (High contrast white on black)**

This mode reverses the positive image. Pictures and text will be displayed as white text on a black background.

- **High contrast white on blue**

This mode displays white text on a blue background.

- **High contrast yellow on black**

This mode displays yellow text on a black background.

- **High contrast yellow on blue**

This mode displays yellow text on a blue background.

- **High contrast green on black**

This mode displays green text on a black background.

Find function

The find function is a useful tool that provides a global view of the document on screen, and that enables you to quickly scroll through the text.

Press this button on the control pad or on the camera head, and hold it down to activate the find function. When the find function is activated, the MultiView will automatically change the level of magnification to the lowest size to allow the global view display. A sign will appear to indicate the centre point of the screen. Move the camera to properly centre your text on screen. Once you have selected your position in the text, release the button. The MultiView will automatically zoom in on the portion of text following your selected position, and will resize to your previous magnification setting.

Autofocus lock

The MultiView has an autofocus feature that allows you to adjust the clarity of the image on screen. When the MultiView is turned on, the system will start up in the autofocus mode.

Press this button located on the control pad to lock the focus. This will stop the MultiView from continuously readjusting the focus. The message “Focus lock” will appear in the upper right-hand corner of the screen. This is useful for tasks such as writing, painting or doing craftwork. Pressing this button will reactivate the autofocus feature, cancelling the message “Focus lock”.

Using the Snapshot button

The snapshot button allows you to take a temporary picture of a text or an image. Press the Snapshot button on the control pad to freeze the image. Pressing any other button will unfreeze the image.

Activating line markers / window shades

Pressing this button on the control pad cycles you through the following options: horizontal line markers, horizontal window shades and full screen camera image. Line markers are used as guides to read text more easily. Window shades are used to block part of the image when the screen is too bright for viewing comfort.

Positioning line markers / window shades

When horizontal line markers or window shades are activated, press these buttons to change the space size between line markers / window shades. To widen the space, press the  button and to narrow it, press the  button.

White Balance

The White Balance function is used to adjust the balance the colors of a picture displayed in photo mode. Press this button on the control pad to cycle you through 2 white Balance modes:

- **Warm color mode**

This mode will enhance the warm colors of a picture such as the color red.

- **Cool color mode**

This mode will enhance the cool colors of a picture such as the color blue .

Left / right rotation

When choosing the camera's position according to hand orientation from left to right, or vice-versa, the camera image will display upside down. Simply press this button to display the camera image rightside up.

6. Contact Information / Product Support

If you have any questions about operating, installing or configuring your system, please contact your MultiView distributor or the Optelec office nearest you. For a complete list of Optelec offices, see the list of addresses on the last page of this user manual.

Your MultiView distributor:

7. Warranty Conditions

Optelec warrants the MultiView, effective from the date of delivery, to be free of any defects in material and workmanship.

Warranty is not transferable and does not apply to groups, multi users or agencies. The MultiView has been designed for the individual purchaser and to be used in a home or office environment. Optelec reserves the right to repair or replace any purchased MultiView with a similar or superior product.

In no event shall Optelec or any of its distributors be held liable for any indirect or consequential damages. This warranty is valid only when serviced in the original country of purchase and if seals are intact. For additional warranty claims or for service during or after warranty period, please contact your distributor.

Optelec takes no responsibility for any other use of this device than the use described in this manual.

8. Compliances

This product complies with the safety and EMC requirements, which fall under the following directives:

Safety

EN 60950-1

EMC

EN 55022

EN 55024

FCC

FCC part 15 Class B

WARNING: The use of accessories, transducers and cables other than those specified and sold by the manufacturer of the MultiView as replacement parts for components, may result in increased emissions or decreased immunity of the MultiView.

This device bears the CE and FCC approval mark.

9. Specifications of the MultiView

Focus	: Autofocus with focus lock
Monitor	: 19" wide screen display (16:10)
Magnification	: 3.5 X – 79 X (on 19" wide LCD screen)
Dimensions	: 55 x 48 x 21 cm
Weight	: Unit: approx 11. 5 kg X-Y table: approx 1.0 kg Unit with XY table and carrying case: approx. 18.3 kg
Viewing modes	: Full color photo mode Gray scale photo mode High contrast black on white High contrast white on black High contrast white on blue High contrast yellow on black High contrast yellow on blue High contrast green on black
Power	: Input 110 - 240 Volt
Net frequency	: 50 - 60 Hz

Operating conditions

Temperature	: +10°C to +35°C
Humidity	: < 70%, no condensation

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Optelec GmbH
Optelec US Ltd
U.S.A.