magnification

ClearNote HD

**User manual
Gebruikershandleiding
Manuel d'utilisation
Manual del usuario
Bedienungsanleitung**

Index

1. Introduction	9
About this manual	9
2. What's in the box?	10
3. Installing the ClearNote HD software.....	10
4. Setting up the ClearNote HD	11
Positioning the camera head.....	11
Camera positions	11
Close-up and distance viewing.....	11
5. The ClearNote HD Toolbar, commands and functions	12
Starting the software	12
The Toolbar.....	12
Auto Focus on / off Button.....	13
Touch screen gestures.....	13
Using the mouse	13
List of commands	13
Adjusting the magnification	15
Viewing modes.....	15
Semi-colors	15
Text thickness	16
Auto Contrast	16
Resizing and moving the ClearNote HD window	16
Activating and sizing line markers	16
Save and open images.....	16
Mirror / flip the image.....	17
Auto Focus on / off	17
6. Using the remote control	18
Basic functions	18
Adjusting the magnification	18
Viewing modes.....	18
Semi-colors	18
Autofocus	18
Shift.....	19
Shift functions.....	19
Switching between open programs	19
Positioning the reading line or the split-screen	19
Refocus.....	19
7. ClearNote HD and magnification software	20
Appendix A: Safety	21
Appendix B: Technical Information	22
Appendix C: Compliances.....	23
Optelec Offices Worldwide.....	91

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User manual

version 1.2

© 2016 Optelec, the Netherlands.
All rights reserved

Optelec
P.O. Box 399
2990 AJ Barendrecht
The Netherlands
Telephone: +31 (0)88 6783 444. Fax +31 (0)88 6783 400
E-mail: info@optelec.nl
Internet: www.optelec.com

1. Introduction

Congratulations on choosing the ClearNote HD, a portable and foldable video magnifier designed by Optelec. The ClearNote HD is a PC connected auto-focus video magnifier to magnify text, photos the blackboard and more, all on your PC screen. The ClearNote HD magnifier can be folded into a compact package so you can easily take it with you.

The ClearNote HD offers an enhanced ergonomic design that allows simple operation. The ergonomics are designed for optimal efficiency when working with a PC. The ClearNote HD can be operated using the PC's touch screen, mouse, keyboard or via the remote control.

The ClearNote HD software is compatible with Windows 8 and higher. The unit should be connected via a USB 3.0 port.

If you have any questions or suggestions concerning the use of this product, please contact your distributor or the Optelec headquarters using the contact information at the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your ClearNote HD.

About this manual

At Optelec, we are constantly improving our products and their functionalities. Therefore, it is possible that this manual is not the latest version. Please download the most up-to-date manual on www.optelec.com at the Support section.

This manual will familiarize you with the features and operation of the ClearNote HD and ClearNote HD software with a PC. Please read this manual carefully before using your device. Keep this manual in a safe place for future reference.

2. What's in the box?

The ClearNote HD packaging contains the following:

- Carrying bag
- ClearNote HD video magnifier
- Base plate
- Remote control
- ClearNote HD software and documentation
- User manual
- Hotkey lists

If any of these items are missing from your package, please contact your local distributor.

3. Installing the ClearNote HD software

Important: Install the ClearNote HD software before connecting your unit to a PC.


To use the ClearNote HD with your PC you will need to install the included software. To install the software:

1. Run the setup;
2. Follow the onscreen setup instructions;
3. Now connect the ClearNote HD USB cable to a USB 3.0 port on your PC and wait until Windows has completed the driver setup;
4. Start the Software by clicking the ClearNote HD 7 icon on your desktop or in the Windows Start Menu.

4. Setting up the ClearNote HD

To setup your ClearNote HD unit please follow the following steps:

1. Place the base plate nearby your PC on a steady surface;
2. Unfold the ClearNote HD unit;
3. Place the ClearNote HD on the connection point of the base plate;
4. Connect the remote control to the camera stand;
5. If you have already installed the ClearNote HD software connect the ClearNote HD output cable (USB 3.0) to your desktop or laptop PC.

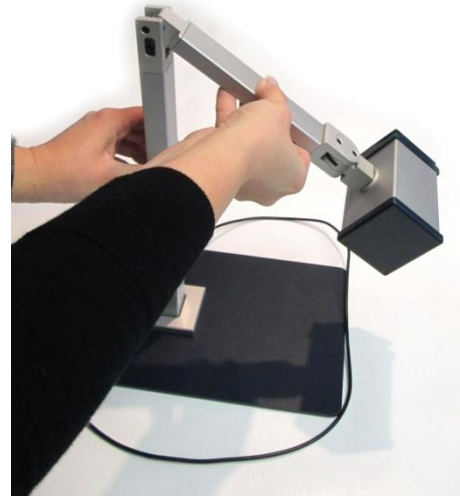When you use a laptop, you can choose to place your laptop on the ClearNote HD base plate. The ClearNote HD can be positioned either left or right of your laptop.

Positioning the camera head

The ClearNote HD features an ultra-flexible camera that can be rotated around a horizontal and vertical axis. Point the camera down to view documents on your desk, or position the camera horizontally for distance or self-viewing. The flexible camera view is more than 180° vertically and horizontally.

Camera positions

The ClearNote HD automatically re-adjusts to its original settings when the camera is turned from near to distance viewing and back. For example, when you tilt the camera quickly from a whiteboard to your desktop, your preferred viewing mode and magnification settings will automatically be applied. Three sets of settings can be stored: document viewing, distance viewing and self view / looking behind you.

Close-up and distance viewing

The position of the camera unit can be adjusted for desktop and distance viewing. To view objects within 25 cm, position the close-up lens in front of the camera unit. To view objects at a larger distance, open the close-up lens so it is no longer in front of the camera.

5. The ClearNote HD Toolbar, commands and functions

Starting the software

Start the ClearNote HD application by clicking either the ClearNote HD icon on your PC's desktop or via the Windows Start Menu.

You can close the ClearNote HD application as any other Windows program using "Alt+F4" or by clicking the close (x) button on the ClearNote HD Toolbar.

The Toolbar

The toolbar contains the following buttons:

Button	Description
	Zoom out
	Zoom in
	Toggle High Contrast / Full Color Photo Mode
	Decrease Window Size
	Increase Window Size
	Open Options Menu
	Open Settings Menu
	Close ClearNote HD Application

Auto Focus on / off Button

Besides the Toolbar buttons, a button with a pencil is being displayed in the bottom left corner of the ClearNote window. Clicking on this icon will enable or disable the Auto Focus of the camera.

Touch screen gestures

You can control the ClearNote HD with touch screen gestures. Please note that touch screen gestures may conflict with your magnifier software depending on the magnifier used and the version of the magnifier software.

The following touch screen gestures are supported:

Gesture	Description
Tap	Left mouse click
Double Tap in Window	Toggle Full Screen ON / OFF
Double Tap on Toolbar	Open Context Menu
Drag Window	Move Window
Pinch Close	Zoom out
Pinch Open	Zoom in
Two Finger Swipe left / Right	Change semi-color combination
Two Finger Swipe up / down	Toggle between photo mode and false color mode

Using the mouse

The following mouse actions are supported:

Mouse Action	Description
Mouse Wheel	Resize Window
Alt+Mouse Wheel	Adjust Line Position
Shift+Mouse Wheel	Adjust Text Thickness in High Contrast Mode
Ctrl+Mouse Wheel	Zoom in / out
Left Double Click	Full screen ON / OFF
Right Click	Context Menu
Right Click on pressed Focus icon	Refocus when Auto Focus is off
Ctrl+Left Click on pressed Focus icon	Refocus when Auto Focus is off

List of commands

The list with hotkeys can be accessed pressing “F1” or by opening the help menu in the Toolbar.

Hotkey	Description
Arrow Down	Zoom out
Arrow Up	Zoom in
Arrow Left	Previous color combination
Arrow Right	Next color combination
Ctrl+Arrow Left	Decrease window size
Ctrl+Arrow Right	Enlarge video window size
Shift+Arrow Down	Change window position vertically (down)
Shift+Arrow Up	Change window position vertically (up)
Shift+Arrow Left	Change window position horizontally (left)
Shift+Arrow Right	Change window position horizontally (right)
Ctrl+Shift+Arrow Down	Decrease Text thickness
Ctrl+Shift+Arrow Up	IncreaseText thickness
Page Down	Line position > Down
Page Up	Line position > Up
F1	Help
F2	Save image
F3	Open image folder
F5	Toggle Semi-color / Full color Photo mode
F6	Smooth text ON/OFF
F9	Hide/Show toolbar
F10	Save image configuration
F11	Video device selection
Enter	Full screen ON/OFF
Escape	Minimize window
Space	Menu
Alt+Space	Settings Menu
C	Auto Contrast Adjustment ON / OFF
F	Toggle Auto Focus ON / OFF
G	Refocus when Auto Focus is off
I	Open Saved Image Folder
L	Toggle Line ON / OFF
Ctrl+M	Horizontal flip ON / OFF
Shift+M	Vertical flip ON / OFF
P	Cycle Window Position
R	Rotation Left / Right / OFF
Ctrl+R	Camera Reset
S	Sharpen ON / OFF
T	Toggle Toolbar Position
V	Always On Top ON/OFF
W	Toggle inverse Toolbar ON / OFF
Shift+Q	Save Image 1
Shift+Ctrl+Q	Open Image 1

Shift+W	Save Image 2
Shift+Ctrl+w	Open Image 2
Shift+E	Save Image 3
Shift+Ctrl+E	Open Image 3
Shift+R	Save Image 4
Shift+Ctrl+R	Open Image 4
Shift+T	Save Image 5
Shift+Ctrl+T	Open Image 5
Shift+Y	Save Image 6
Shift+Ctrl+Y	Open Image 6
Shift+U	Save Image 7
Shift+Ctrl+U	Open Image 7
Shift+I	Save Image 8
Shift+Ctrl+I	Open Image 8
Shift+O	Save Image 9
Shift+Ctrl+O	Open Image 9
Shift+P	Save Image 10
Shift+Ctrl+P	Open Image 10
A+Q	Show Application Settings
X	View Zoom Area
Ctrl+ Win+Space	Application Window On Top
Win+F2	Full screen mode ON / OFF

Adjusting the magnification

Click on the “-“ button to decrease the magnification. Click on the “+” button to increase the magnification.

Pressing the “Arrow down” button will decrease the magnification. Pressing the “Arrow up” button will increase the magnification.

Use the pinch commands to increase and decrease the magnification.

Viewing modes

You may choose to display text in a pair of selectable colors when more contrast is needed or when high contrast color combinations provide better visibility. Click on the “Mode” button or press the “F5” button on the keyboard to toggle between Full color photo mode and high contrast semi color reading mode. Alternatively, use the touch screen gesture swipe up or down with 2 fingers.

Semi-colors

You may choose to display text using a range of high-contrast semi-color combinations. These can be activated by clicking on the “Mode” button or

pressing “F5” to switch to High Contrast mode. To cycle through each of the different foreground and background colors use the left and right arrow keys or a swipe left or right with two fingers on the touch screen.

Text thickness

To improve the readability of the text in high contrast mode, the text thickness can be adjusted using Control+Shift+Arrow Up and Down.

Auto Contrast

When enabled, the Auto Contrast function will attempt to eliminate shadows and light spots in the ClearNote HD image. This function can be helpful in full color mode and in poor light conditions. When one of the semi color modes are enabled, and in normal light conditions, it is highly recommended to leave this function off. The hotkey to toggle Auto Contrast on or off is “c”. By default, Auto Contrast is on in full color mode and off in semi color mode.

Resizing and moving the ClearNote HD window

The size of the ClearNote HD window can be adjusted by clicking on the two window resizing buttons on the Toolbar.

Press “Enter” or “Win+F2” to restore the ClearNote HD window, or to switch to full screen mode. “Escape” will minimize the ClearNote HD window.

The ClearNote HD window can be repositioned by pressing “Ctrl+Win+Space”.

Activating and sizing line markers

The reading line can be used for better orientation or as a reference when using the ClearNote HD. Press “L” to activate the reading line. Press “Page up” or “Page down” to move the line up or down.

Save and open images

To save an image press “F2”.

To open the folder with saved images press “F3” or “i”.

Images are named using a prefix and number in a numerical order. The folder to which the images are saved as well as the used prefix can be set in the “Save Frame Dialog”. To open the “Saved Frame Dialog”, click on the Settings button (Alt+Space) in the Toolbar and select “Frame Saving Configuration” from the menu. Here you can set:

- The destination folder
- The Image File format

- The File prefix
- Number format for existing files

Mirror / flip the image

Pressing “Shift+M” allows you to horizontally mirror / flip the video image. To mirror the image vertically, press “Ctrl+M”. You may choose this function when you have tilted the camera for self-viewing, because self-viewing results in an image that is upside down. Alternatively you can rotate the image by pressing “R”.

Auto Focus on / off


The ClearNote HD features automatic focusing. This produces a clear image at any time. Turning Auto Focus off, stops the ClearNote HD from continuously focusing. This is useful for tasks such as writing, painting or enjoying craftwork.

To toggle Auto Focus on or off, select the Pencil icon located in the bottom left corner of the ClearNote HD window. Alternatively, press the pencil button on the Remote Control or F on the keyboard.

If the Auto Focus is off, you can manually refocus the camera by pressing G, Control + left mouse click, right click, or pressing Shift + Focus on the Remote Control.

6. Using the remote control

Basic functions

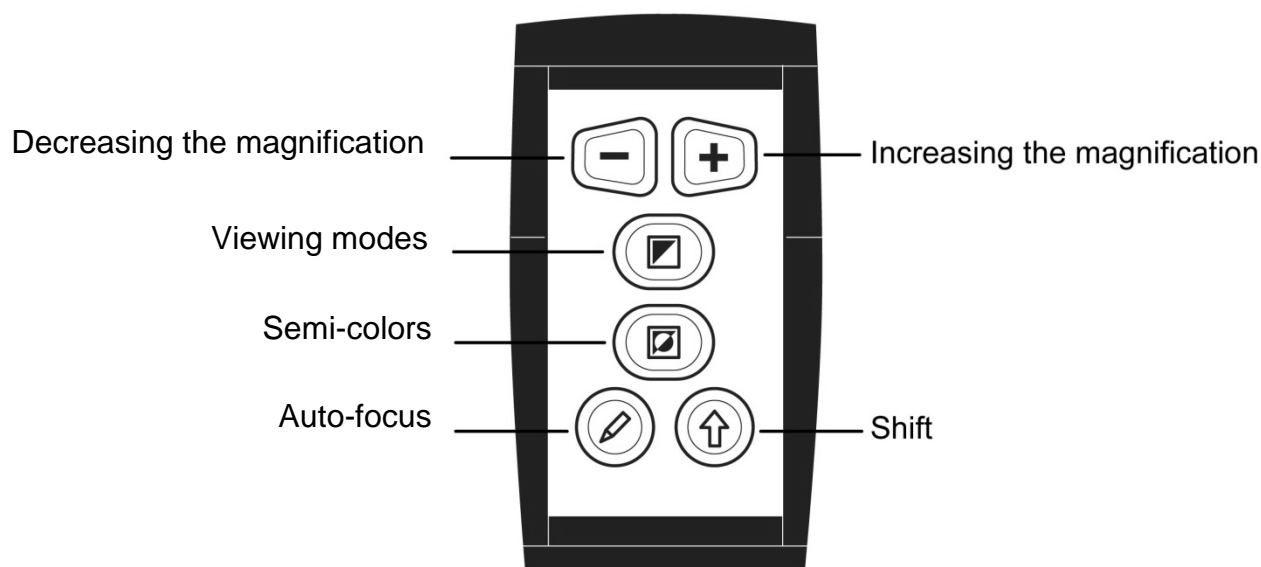Adjusting the magnification

Pressing the two trapezium shaped buttons allows you to change the magnification. Pressing the “-” button will decrease the magnification. Pressing the “+” button will increase the magnification.

Viewing modes

Pressing this button will toggle semi color mode on / off.

Semi-colors

When high contrast semi color mode is enabled, Pressing this button cycles you through 16 different foreground and background color combinations.

Autofocus

Press this button to disable or enable Auto Focus.

Autofocus on


Autofocus off

Shift

Pressing the Shift button will activate the ClearNote HD window. Pressing this button in combination with other buttons performs additional functions. Read the following page about the Shift functions.

Shift functions

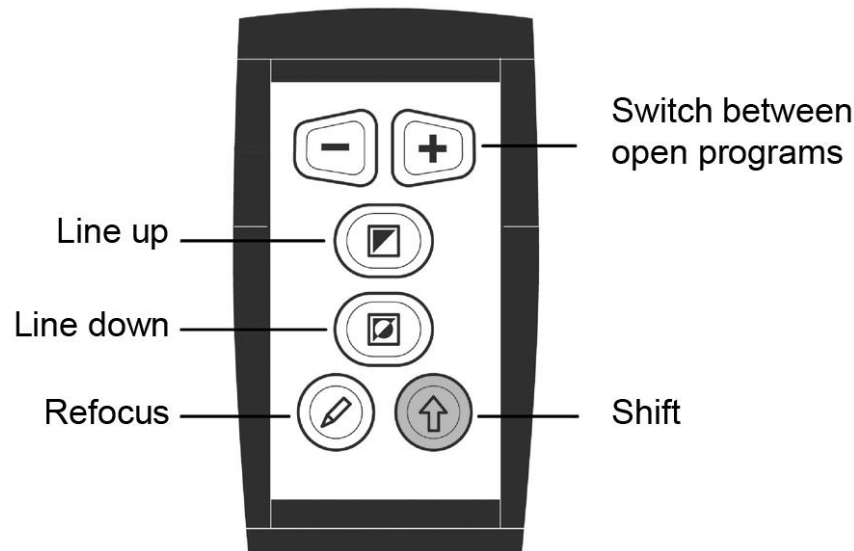Switching between open programs

Pressing the “Shift” and the “+” button simultaneously to switch between the open applications.

Positioning the reading line or the split-screen

Press the “L” key on the PC keyboard to add a reading line to the ClearNote HD image. Press the “Shift” and “Viewing mode” button simultaneously to move the line up. Press the “Shift” and “Semi-colors” button to move the line down.

If the ClearNote HD is used together with magnification software, the ClearNote HD window and Magnification window will be displayed side by side, in a split-screen format. In this case pressing the “Shift” and “Viewing mode” button simultaneously will move the split up or left. Pressing the “Shift” and “Semi-colors” buttons the split down will be moved down or right.

Refocus

Press “Shift” and “Focus” to refocus the camera when Auto Focus is disabled.

7. ClearNote HD and magnification software

The ClearNote HD can be used simultaneously with magnification software programs such as SuperNova and ZoomText.

The ClearNote HD image appears adjacent to the magnified Window, in a split-screen. The ClearNote HD image can be re-sized and positioned anywhere on your screen. The split screen will be automatically adjusted.

The ClearNote HD software will detect Zoomtext or Supernova automatically. Start your magnification software first, followed by the ClearNote HD application. When closing your magnifier, make sure to close the ClearNote HD application first.

In the About window, the detected magnification software is being displayed. To open the about screen, select Settings (Alt + Space) in the Toolbar.

Support for magnifiers is limited to Dolphin Supernova version 14.04 or higher and ZoomText version 10.1 or higher.

Appendix A: Safety

- Only use the ClearNote HD indoors.
- Do not expose the ClearNote HD to extreme or rapid changes in temperature or humidity. These conditions may damage the unit's electronics or optics.
- Do not place the unit on an unstable surface. It may fall, causing serious damage to the unit or injuries to the user.
- To avoid injuries, always assemble, disassemble and handle the unit with good care as the unit contains movable and rotatable parts.
- Do not expose the ClearNote HD to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the ClearNote HD. Contact your Optelec distributor to service the ClearNote HD if necessary. Do not open your unit as this will void the warranty. The unit contains no user-serviceable parts.
- To avoid risk of electrical damage, keep your ClearNote HD away from fluids and chemicals
- Handle the ClearNote HD with care. Rough handling will damage the internal components
- Do not use the ClearNote HD near inadequately shielded medical devices.
- Always unplug the unit before cleaning. Use a soft damp cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage the unit.
- Do not use the unit under 0° or above +40° Celsius.

Use of the ClearNote HD other than described in this manual will exclude it from warranty conditions.

Appendix B: Technical Information

- Camera : 12x optical zoom
- Viewing modes : Full color photo mode
High contrast black-on-white
High contrast white-on-black
- Semi-colors : 16 foreground / background
color combinations
- Magnification : 1.3 to 40 times on a 15" monitor
- Focus : Autofocus with focus lock
- Weight : 1,4 kg
- PC requirements : Windows 8 or higher
- Connections : USB 3.0

Important: USB 3.0 ports using the Renesas μ PD720200A chipset are not supported.

Operating conditions

- Humidity : < 75%, no condensation
- Temperature : 0 – 40 °C

Storage & transport conditions

- Humidity : < 75%, no condensation
- Temperature : -20 – 60 °C

WARNING: The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by the manufacturer of the ClearNote HD as replacement parts for internal components, may result in increased emissions or decreased immunity of the ClearNote HD.

Optelec nor its partners do not take any responsibility when using the product and its accessories other than those specified in this manual.

For additional questions on the ClearNote HD regarding service, installation or operation, do not hesitate to contact your local distributor or closest Optelec office. See the last pages of this manual for a complete list of Optelec addresses.

Appendix C: Compliances

This product complies with the safety and EMC requirements of the directives:

EN60601-1 : 2006

EN60601-1-2 : 2007

Optelec

Breslau 4

2993 LT Barendrecht

The Netherlands

EU – Declaration of conformity

2014, Barendrecht, the Netherlands

The manufacturer declares that the:

Video magnifier **ClearNote HD** complies to the safety guidelines and directives of the European Community.

This EU – declaration of conformity is based on tests according to the directives:

EN 60601-1:2006 and EN 60601-1-2:2007 by an independent Notified body.

This device bears the CE approval mark

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Optelec GmbH